



MEDI-FLO[®]

ELASTOMERIC PUMP

MEDI-FLO[®] Elastomeric Pumps provide a drug delivery system that is simple and safe to use for pharmacists, nurses and patients. These guidelines reflect delivery times for partial or overfill volumes. To find partial or overfill volume for a particular model, locate the desired delivery time in the far left column and slide to the right to the column referencing the item code to find fill volume. Please note the delivery times for partial or overfill volume are approximate values. Please reference the product instructions for use of the MEDI-FLO[®] for additional information.

Filling Guide

Short Duration Pumps

Item Code	MDE050500	MDE100500	MDE101000	MDE102000	MDE200100	MDE200200	MDE250050	MDE250100	MDE250125	MDE251750	MDE252500	MDE250500	MDE271750	MDE401000	MDE402000	MDE502500
Nominal Fill Volume	50 mL	100 mL	100 mL	100 mL	200 mL	200 mL	250 mL	250 mL	250 mL	250 mL	250 mL	250 mL	270 mL	400 mL	400 mL	500 mL
Nominal Flow Rate	50 mL/H	50 mL/H	100 mL/H	200 mL/H	100 mL/H	200 mL/H	50 mL/H	100 mL/H	125 mL/H	175 mL/H	250 mL/H	500 mL/H	175 mL/H	100 mL/H	200 mL/H	250 mL/H
Min Fill Volume	50 mL	75 mL	50 mL	50 mL	150 mL	150 mL	150 mL	200 mL	150 mL	140 mL	150 mL	250 mL	250 mL	275 mL	200 mL	360 mL
Max Fill Volume	60 mL	110 mL	120 mL	100 mL	300 mL	300 mL	275 mL	300 mL	300 mL	300 mL	300 mL	300 mL	340 mL	550 mL	500 mL	550 mL
Residual Volume	2 mL	2 mL	2 mL	2 mL	2 mL	2 mL	3 mL	3 mL	3 mL	3 mL	3 mL	3 mL	3 mL	5 mL	5 mL	5 mL

Approximate Delivery Time (Hours:Min)																
0:15				50 mL												
0:30			50 mL	100 mL							150 mL	250 mL				
0:45			75 mL			150 mL				140 mL	200 mL	300 mL				
1:00	50 mL		100 mL			200 mL				175 mL	250 mL				200 mL	
1:15	60 mL		120 mL			250 mL			150 mL	220 mL	300 mL				250 mL	360 mL
1:25													250 mL			
1:30		75 mL			160 mL	300 mL			180 mL	250 mL			270 mL		300 mL	400 mL
1:45					180 mL				220 mL	300 mL			300 mL		350 mL	475 mL
1:55													320 mL			
2:00		100 mL			200 mL			200 mL	250 mL				340 mL		400 mL	500 mL
2:15		110 mL			220 mL			220 mL	280 mL						450 mL	550 mL
2:30					240 mL			250 mL	300 mL						500 mL	
2:45								275 mL								
3:00							150 mL	300 mL						275 mL		
3:30														300 mL		
4:00														360 mL		
4:30														400 mL		
5:00								250 mL						450 mL		
5:30								275 mL						500 mL		
														550 mL		

PLEASE NOTE:

1. Delivery times for partial or overfill volume are approximate values.
2. Filling the pump more than the nominal fill volume results in a slower flow rate.
3. Filling the pump less than nominal fill volume results in a faster flow rate.
4. Do not fill the pump less than the minimal or more than the maximum fill volume specified on the chart above.
5. Please refer to the Instructions for Use for further information on use of the MEDI-FLO[®] Elastomeric Pump.



MEDI-FLO[®]

ELASTOMERIC PUMP

MEDI-FLO[®] Elastomeric Pumps provide a drug delivery system that is simple and safe to use for pharmacists, nurses and patients. These guidelines reflect delivery times for partial or overfill volumes. To find partial or overfill volume for a particular model, locate the desired delivery time in the far left column and slide to the right to the column referencing the item code to find fill volume. Please note the delivery times for partial or overfill volume are approximate values. Please reference the product instructions for use of the MEDI-FLO[®] for additional information.

Filling Guide

Long Duration Pumps & Chemotherapy Pumps

Item Code	LONG DURATION					CHEMOTHERAPY				
	MDL100002	MDL270002	MDL270005	MDL270010	MDL500020	MDC100002	MDC270010	MDC270020	MDC270050	MDC300060
Nominal Fill Volume	100 mL	270 mL	270 mL	270 mL	500 mL	100 mL	270 mL	270 mL	270 mL	300 mL
Nominal Flow Rate	2 mL/H	2 mL/H	5 mL/H	10 mL/H	20 mL/H	2 mL/H	10 mL/H	2 mL/H	5 mL/H	6 mL/H
Min Fill Volume	75 mL	150 mL	250 mL	230 mL	360 mL	75 mL	230 mL	150 mL	250 mL	250 mL
Max Fill Volume	110 mL	300 mL	360 mL	360 mL	500 mL	110 mL	360 mL	300 mL	360 mL	360 mL
Residual Volume	2 mL	3 mL	3 mL	3 mL	5 mL	2 mL	3 mL	3 mL	3 mL	3 mL

Approximate Delivery Time											
Hours:Min	Day(s)	MDL100002	MDL270002	MDL270005	MDL270010	MDL500020	MDC100002	MDC270010	MDC270020	MDC270050	MDC300060
6:00											
8:00											
10:00											
12:00											
12:30											
16:00											
18:00						380 mL					
20:00					230 mL	440 mL		230 mL			
25:00	1				250 mL	500 mL		250 mL			
27:00	1				270 mL			270 mL			
30:00	1				300 mL			300 mL			
36:00	1.5	72 mL			330 mL		72 mL	330 mL			
40:00	1.5	80 mL					80 mL				250 mL
46:00	1.5	96 mL		240 mL			96 mL			240 mL	280 mL
50:00	2	100 mL		250 mL			100 mL			250 mL	300 mL
54:00	2	108 mL		270 mL			108 mL			270 mL	320 mL
62:00											360 mL
72:00	3			330 mL						330 mL	
75:00	3										
80:00	3		200 mL						200 mL		
100:00	4		220 mL						220 mL		
125:00	5		250 mL						250 mL		
135:00	6		270 mL						270 mL		
144:00	6		300 mL						300 mL		
150:00	6										
168:00	7										

PLEASE NOTE:

1. Delivery times for partial or overfill volume are approximate values.
2. Filling the pump more than the nominal fill volume results in a slower flow rate.
3. Filling the pump less than nominal fill volume results in a faster flow rate.
4. Do not fill the pump less than the minimal or more than the maximum fill volume specified on the chart above.
5. Please refer to the Instructions for Use for further information on use of the MEDI-FLO[®] Elastomeric Pump.